


Unique orthopaedic shoes for the treatment of lameness in cattle

- Cowslips™ are easy, quick and clean to fit
- Mixing the fast curing adhesive in the shoe permits cleaner, easier and faster application within minutes
- Self-shedding in 4-6 weeks
- Cost effective
- Hoof-like design, right and left fit
- Fitted to the healthy claw, Cowslips™ (with a raised non-slip latticed sole) speeds up healing by taking the weight off the damaged claw
- Increases mobility, thus improving milk yield and weight gain
- Manufactured from recycled PVC



1

Trim and thoroughly clean the claw.
Claw must be dry;
wipe with 'Acetone' or
'Methylated Spirit'.

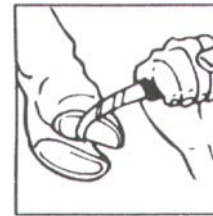


2

Trial fit the shoe before
applying adhesive.
Some trimming of the
claw may be
necessary.



3

Thoroughly mix the
adhesive within the
shoe to achieve an
even paste
consistency.

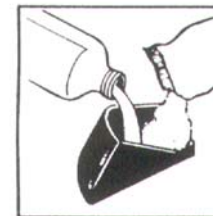

4

Fit the shoe on healthy
claw



NO MORE LAME EXCUSES!

5

Position firmly by
tapping the toe of the
shoe.
*NB. This is important to
achieve the ideal positioning.
Sole of the shoe should be
parallel to sole of claw.*




6

Clean off any excess
adhesive.




7

Leave foot in raised
position for several
minutes until adhesive
has set.

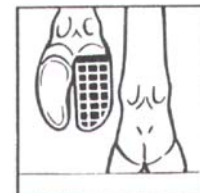

8

Place foot on ground
and keep the animal
standing for a few
minutes.

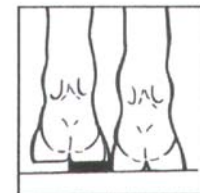

9

Animal is now free to
walk uninhibited

